# MelkVerband

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August 2011

# Content

<b>Product features</b>	<b>16</b>
<b>Advantages</b>	<b>17</b>
<b>Dressing Configuration</b>	<b>18</b>
Milking Bandage	18
Teat tip protection	19
<b>Application instructions</b>	<b>20</b>
Cleaning of the teat	20
Preparation of the dressing	21
Application of the dressing	21
Check that the dressing is stuck in place before and after every milking	22
Removal of the dressing	22
<b>Further details</b>	<b>23</b>
<b>Information about the Milking Bandage</b>	<b>24</b>

**This is a self-adhesive dressing that provides effective protection for stressed teats and open wounds on teats.**

Please carefully read the instructions before use. Please direct inquiries to the manufacturer or to your veterinarian.

## Product features

- The dressing provides effective protection for open wounds on teats, as well as stressed teats during milking and in the stall.
- The dressing helps the healing process by providing a moist environment for wound healing.
- Multiple milkings are possible without changing the dressing.
- Can also be used to increase teat diameter for fitting milking equipment, by applying several bandages, one overlapping the other.


## Advantages

- The animal is remarkably calmer when using the Milking Bandage. Normal milking is possible.
- The dressing protects the teat during the milking process.
- The dressing acts like a natural protective layer to the wound, preventing hard scab formation.
- Hardening of the tissue will soften after about 24 hours. This will allow normal emptying of the quarter.
- Existing hard scabs will soften and disintegrate after about 24 hours.
- The damaged area will remain moist until the wound closes to encourage natural healing.
- A moist wound environment encourages the healing process and enables faster healing.
- The dressing does not stick to the wound surface, therefore preventing damage upon removal of the dressing.
- The teat remains smooth and soft after the healing period.
- The dressing adheres well and remains smooth and flexible even in low temperatures.
- Teat dipping is possible.

*A comparison against treatment without Milking bandage you can find on [www.Dr-Kenndoff.eu](http://www.Dr-Kenndoff.eu):  
"Products – Milking bandage – Advantages"*

## Dressing Configuration


### Milking Bandage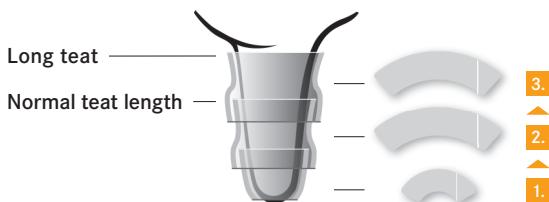

Apply the dressing by starting at the teat tip and work towards the base of the udder. The dressing configuration depends on the size of the teat and the size of the wound. A dressing with the size 29 x 122 mm applied on the teat tip always must be fixed with a dressing 38 x 176 mm.

Teat diameter can be increased for fitting milking equipment, by applying several bandages, one on top of the other.

Trim the dressing where necessary.

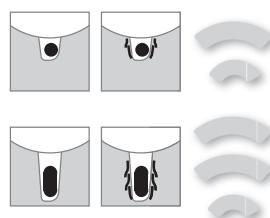### Teat tip protection

The teat tip section should be applied last, to enable that part of the dressing to be removed at any time and replaced.

The teat tip section must be removed before milking, and can be replaced by a new tip layer after milking.

For additional Information on how to use the product go to  
[www.Dr-Kenndoff.eu](http://www.Dr-Kenndoff.eu):  
 "Products - Milking bandage - Dressing configuration".

## Application instructions

- Restrain the animal suitably to prevent it kicking.
- The dressing is applied to the teat before milking the animal.
- Take care not to stretch the dressing!
- Make sure that the dressing fully overlaps at the seam (about 50%).

### Cleaning of the teat


- Stem bleeding.
- To close wound gaps.
- Do not remove a scab.
- Disinfect the wound if necessary.

**Always:** use a moist cloth or tissue to carefully clean the wound and teat from adherent dirt.

**Teat cleaning with a robot requires additional fixation with „Fixation strip Milking bandage“.**


## Preparation of the dressing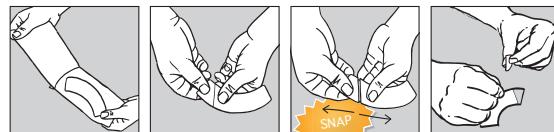

- Ensure that you have clean and dry hands.
- Do not touch the adhesive surface.
- It is advisable to use clean dry hands without gloves, as gloves might handicap the correct application of the dressing.

## Application of the dressing


- The teat skin must be dry and clean.
- Avoid wrinkles while applying the dressing.
- **The curve must point downwards!**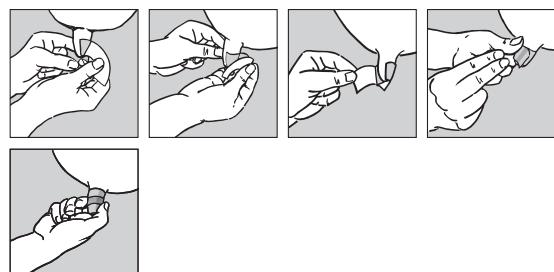

## Further details

- Pull off the paper away from you and around the teat.
- Brush the dressing with your hand as even as possible to smooth out any wrinkles.
- Animals kept outdoors n wet weather will require additional fixation „Fixation strip Milking Bandage“.**
- Check that the dressing is stuck in place before and after every milking**
- Carefully clean the dressing with a moist cloth or tissue.
- If the dressing rolls off while the teat cup while being removed, it is normally easily rolled back into place.
- During the first days of treatment with Milking bandage a yellow serous fluid can be present in large amounts. This is an normal healthy reaction.

## Removal of the dressing

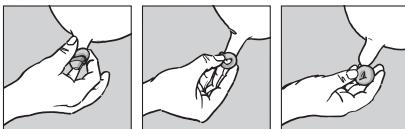For additional formation on how to apply the product, go to  
[www.Dr-Kenndoff.eu](http://www.Dr-Kenndoff.eu): "in Products - Milking bandage - Application instructions".

Additional Information you can find on [www.Dr-Kenndoff.eu](http://www.Dr-Kenndoff.eu).

## Information about the MelkVerband

### Composition

Self-adhesive Polyurethane, Natriumpolyacrylate, without actives.

### Product sizes

1,5 x 6,9 inch

1,1 x 4,8 inch

### Material- or Indication group

Self-adherent wound dressing for treating and protecting teat injuries.

### Pharmaceutical company

Dr. Kenndoff GmbH & Co. KG,  
Tempowerkring 6,  
D-21079 Hamburg,  
Germany  
Telephone: +49 40 79 01 21 10

### Field of application

External use for several days on skin and wound.

### Contraindication

Do not use on infected wounds.

### Precautions while using the dressing

- Do not stretch the dressing while applying it.
- If you apply multiple dressings, always do so before milking.

### Interference with other products

Not applicable.

### Warnings

- If applied on intact teats, avoid dipping the dressing.
- Keep product out of reach of children.

### Nature and length of application

The Bandage can remain on the teat as long as it is clean and protects.  
Dressing must be changed after 7 days.

### Side effects

None known.

### Latency

None.

### Shelf life

See packaging.

### Storage conditions

Store in dry conditions and avoid exposure to sunlight.

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