Mortella Heal

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English

Content

Product features	20
Advantages	21
Applying the bandage	22
Fix the claw	22
Clean the claw	23
Apply Mortella Heal	23
Pad the claw	25
Secure Mortella Heal and gauze	27
Further Information	30
Information about Mortella Heal	32

Product features

Mortella Heal covers **Digital Dermatitis** lesions like a second skin to protect against contact with dirt, bacteria and moisture from the outside while being permeable to oxygen and moisture. The dressing absorbs excess wound fluid and concentrates the body's own wound fluids on the surface of the wound.

Therefore, **Mortella Heal** provides a very good basis for body's own healing process. Even very large wounds may heal within 14 days and form fresh, light-colored, flexible, and pain-free skin.

Advantages

- Without inhibitor
- Without active ingredient
- No withdrawal period
- Protects the wound like a scab
- Moist wound healing supports body's own healing process
- Supports the formation of fresh, light-colored, flexible, and pain-free skin
- Can help to suppress excessive growth of granulation tissue
- Skin-friendly
- Able to cover large wounds areas
- Can be used on young animals
- Can cause fast or immediate pain relief on application leading to the animals' increased mobility
- A single application is often sufficient
- Another Mortella Heal bandage application can be applied immediately after a previous one

Please read the instructions carefully before use. Please direct inquiries to the manufacturer or your veterinarian.

More information related to Mortella Heal please find on www.Dr-Kenndoff.eu.


Applying the bandage

The description below is described for right-handed application and can be reversed. Where necessary you encompass the claw with your left hand. Your fingers touch the coronary band on the front side and the thumb touches the ball. This allows you to hold the gauze and bandage in position while applying it.

Fix the claw


Secure the claw using appropriate restraint, making the claw accessible.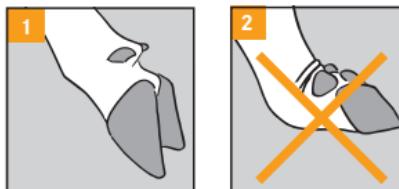

Ensure that the claw is stretched (1) while applying the bandage.

Clean the claw

- Clean the claw and affected areas (3).
It is recommended to clean the claw with lukewarm water e.g. from an udder shower. After that, you can be sure to see smaller lesions.
- Inspect the interdigital space and the coronary band on the front side of the claw for Digital Dermatitis wounds.
- Cut off long hairs.
- Dry the claw using tissue paper.

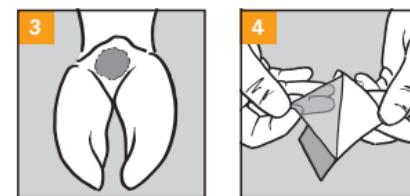
Apply Mortella Heal

- Remove the release liner from **Mortella Heal**.
Remove the release liner starting from an edge (4) or by cracking the perforation.

- Put the sticky surface of the product onto the wound. Cover the wound completely (5) if necessary by using several dressings overlapping.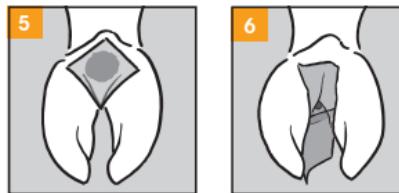

Example: the ball of the foot (5).

In the same way, apply **Mortella Heal** dressing on wounds of the coronary band on the front side of the claw.


Cover Digital Dermatitis wounds in between the claws with two **Mortella Heal** dressings (72 x 72 mm) overlapping (6). Stick together the two dressings before you apply it. As an alternative use a dressing specifically designed for this purpose (162 x 86/50 mm).

Ensure that you cover all Digital Dermatitis wounds around the foot, if necessary with several dressings, before you apply the bandage!

Pad the claw

You need half a roll of the gauze provided.

- Put three layers of the gauze on top of each other (7) (three-ply for around 20 cm) and twist it two times (8) near to the gauze roll in your left hand.

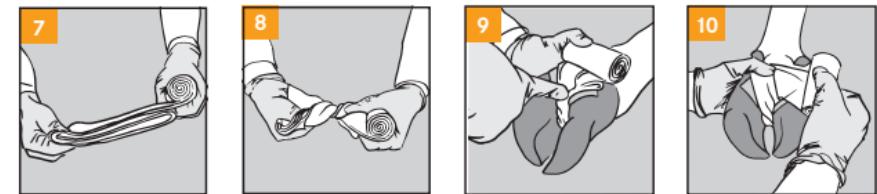
- Place the gauze roll on top of the dressing and pull the twisted gauze into the interdigital space.
- Secure the gauze with the thumb on the ball and with the fingers on the coronary band in front of the claw.
- Press the twisted gauze with a finger of your right hand deep into the narrow part of the interdigital space next to the ball (9).

- Pull the gauze with your right hand so that the gauze fits close to the interdigital space and to the coronary band in front of the claw.

Inspect whether the dressing still covers the wound(s) completely.

- Take the gauze roll into your right hand and turn the gauze 90° to the right (10).
- Wrap the gauze once around the leg, so that the gauze upholsters the complete coronary band while securing the beginning of the gauze.
- Place the pressure pad pyramid provided onto the gauze on top of the dressing.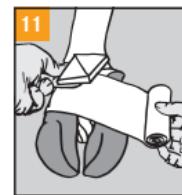




- Wrap the gauze 1 1/2 times around the leg, so that it holds the pyramid in position and the gauze upholsters the leg right up to the dewclaw.
- After that tear off the gauze.

Secure Mortella Heal and gauze

You need half a roll of the self-adhesive elastic bandage provided.

Apply the bandage with appropriate tension, where possible with the entire width of the bandage, so that the bandage does not cut off the blood supply.

- Place the base of the pyramid onto the gauze on top of the dressing (11), if the wound is located on a proliferation.
- Place the tip of the pyramid onto the gauze on top of the dressing (12), if the wound area is depressant.

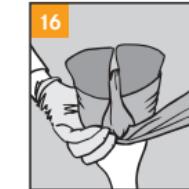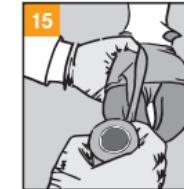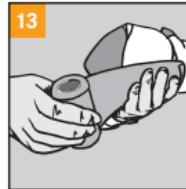- Wrap the beginning of the bandage from front to back in-between the gap of the claw. Hold the bandage with the fingers of your left hand on the coronary band (13).
- ... and with the thumb on the ball (14).
- Turn the bandage by 90° to the right and wrap it twice around the foot (14).
- Hold the bandage again with the thumb of your left hand on the ball (15).
- Turn the bandage by 90° to the right and wrap it from back to front in-between the gap of the claw **along the left wall** of the claw (15).
- Hold the bandage with the fingers of your left hand on the coronary band and turn the bandage by 90° to the right (16).

- Wrap the bandage back to the ball and hold it again with the thumb of your left hand (17).
- Turn the bandage by 90° to the left and wrap it from back to front in-between the gap of the claw **along the right wall** of the claw (17).
- Turn the bandage on the coronary band by 90° and wrap the bandage 1 1/2 times around the foot.
- Tear off the bandage if necessary and push the end behind the edge of the bandage (18).
- Smash the border, so that the edge of the bandage cannot cut into the foot (18).
- Secure the bandage with two wraps of a duct tape (19).

The bandage remains 14 days (± 3) days on the claw.

Further Information

Supports

- It is essential to fix Mortella Heal with the bandage.
- A small amount of residual dirt on the Digital Dermatitis wound or the dressing does not reduce the healing process.
- A professional bandage must ensure that,
 - Mortella Heal is in contact with the wound surface or the wound fluid.
 - dirt doesn't penetrate from outside to the wound.
 - the bandage doesn't cut into the flesh.

Precautions

- Any Digital Dermatitis wounds that you do not cover completely may deteriorate. This becomes apparent if the animal still shows pain reaction after about two to three days. Then you must clean and inspect the claw again and replace the bandage with a new one.
- Remove the bandage in case of a swollen foot and consult your veterinarian.
- Do not cover phlegmons with a bandage.

Disposal

- Dispose of the used product with domestic waste.
- The product is not compostable.

Other

- Strictly for animal use only.
- European and single state laws, rules and regulations have to be strictly observed while using the dressing.

Information about Mortella Heal

Composition:

Self-adhesive polyurethane; Sodiumpolyacrylat; without actives.

Product Size:

2.82 x 2.82 inch

6.38 x 3.39/1.97 inch

Material or indication group

Wound dressing to cover Digital Dermatitis wounds to support the self-healing process.

Pharmaceutical Company

Dr. Kenndoff GmbH & Co. KG,

Tempowerkrieg 6,

D-21079 Hamburg, Germany

Tel.: +49 40 79 012110

Field of application

External use for several days on skin and wound.

Contraindication

Do not use on Plegmons.

Precautions while using the dressing

To fix **Mortella Heal** apply the bandage in a way so that the edge does not cut into the flesh.

Interference with other products

Not applicable.

Warnings

Ensure that the fixing bandage does not cut into the foot. Keep away from children.

Nature and length of application

Clean affected areas from resting dirt. Apply the dressing with the sticky surface to the wound so that it covers a lesion completely. If necessary, use several dressings overlapping. Ensure contact of the dressing to the wound surface or wound fluid. Fix the dressing so that it securely protects the lesion for 14 days. Remove the bandage after 14 days.

Side effects

None known

Latency

None

Shelf life

See packaging

Storage conditions

Store in dry conditions and avoid exposure to sunlight.

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